

THE 100% NON-SURGICAL SOLUTION



LOSE WEIGHT WITHOUT SURGERY



Lose 3x the weight of
diet & exercise alone

Now with 12 months of support

START LOSING 3X THE

TIRED OF SPENDING MONEY ON WEIGHT LOSS PROGRAMS THAT DON'T WORK?

The ORBERA365™ weight loss system provides you with positive results in only 6 months and stays in place for up to 12 months for ongoing support.

ORBERA365™ combines the leading gastric balloon technology along with a support team of experts and a personalized diet and exercise plan all in one program.


This comprehensive program combines the portion control of the stomach balloon with the long-lasting healthy lifestyle coaching to provide effective and safe weight loss.

WEIGHT WITH ORBERA365™




HOW ORBERA365™ WORKS


THE 100% NON-SURGICAL SOLUTION

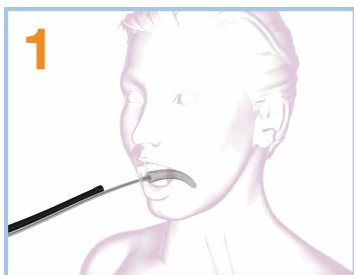

Deflated ORBERA® balloon is inserted through your mouth and into your stomach.

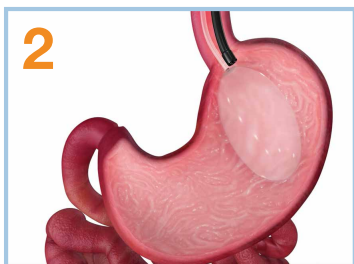


A 50cc syringe is then used to fill the balloon with a sterile saline solution to the size of a grapefruit.

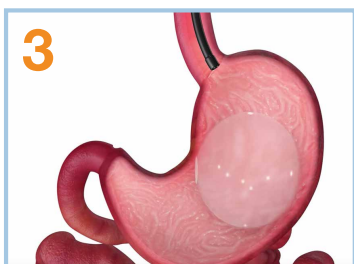

The balloon takes up enough space in your stomach to help you adapt to healthier portion sizes.

UP TO 12 MONTHS LATER

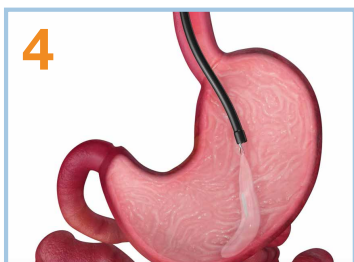

The balloon is removed 12 months later with a non-surgical procedure.

ORBERA365™ HELPS YOU FEEL FULLER AND EAT SMALLER MEALS.

**12
MONTHS**

12 MONTH PLACEMENT & ONGOING SUPPORT

Over a period of 12 months, learn how to change your lifestyle and how to keep the weight off with the care and support of a team of experts.



NON-SURGICAL WEIGHT LOSS

ORBERA365™ is a reversible non-surgical weight loss procedure, which means no incisions, stitches, scars, or anatomical modifications.

277K

REAL WORLD RESULTS

Over 277,000 gastric balloons distributed worldwide.

**20
YEARS**

#1 WEIGHT LOSS GASTRIC BALLOON

ORBERA® is the global leading weight loss balloon and has helped thousands meet their weight loss goals for over 20 years.

OPTIMIZED BALLOON DESIGN

ORBERA® has a reputation built on quality, reliability and the #1 gastric balloon around the world. Key features of the ORBERA® gastric balloon contribute to its excellent standing and years of success. You can feel confident that your gastric balloon has a design focused on your safety and weight loss.





Ultra Smooth and Round

The ORBERA365™ balloon has no protrusions, no sharp edges and no hard surfaces that can irritate your stomach. The innovative ORBERA365™ design offers an incredibly smooth, soft and round shape to put your safety first and limit the risk of stomach ulcerations.



7+ Layers of High Density Silicone

Each ORBERA365™ gastric balloon is created with 7+ layers of some of the highest medical-grade silicone to ensure durability, so it can withstand digestive fluids and stomach acid with ease over the course of 12 months.



Effective Weight Loss

The ORBERA365™ balloon is designed to take up enough space in the stomach to help improve portion control and help you feel full but not uncomfortable. ORBERA365™ balloon size and shape have been shown to help patients lose 3x the weight of diet & exercise.



Optimal and Customizable Size

ORBERA365™ balloon can be tailored to your stomach and take up just the right amount of space. At about the size shape and size of a grapefruit, each balloon can be filled with as little as 400ml or as much as 700ml based on your stomach.

YOUR QUESTIONS ANSWERED

How much does it cost?

The total cost of the ORBERA365™ Program will vary but is typically an all inclusive price that includes the ORBERA® Balloon, Placement Procedure, Removal Procedure, Facility Fees and 18-24 months of physician and dietary coaching support.

What does it feel like?

ORBERA365™ takes up space in patients' stomachs and it will make you feel very full with a small meal. It helps you learn proper portion control over the 12 months with it in your stomach.

It typically takes a few days for your body to adjust to the ORBERA365™ balloon and most patients experience some nausea, vomiting and reflux, but this typically goes away and is managed by your physician.

Does it really work? How much can I expect to lose?

ORBERA® has helped thousands of patients around the world lose that stubborn fat. ORBERA® has been studied in over 200 clinical studies around the world. In the FDA approval study, patients on average lost 3x the weight of diet or exercise, or about 10% of their total body weight.

When will I see the results?

The weight loss is rapid with most of the weight loss occurring in the first three months after balloon placement. ORBERA365™ can be kept in place for up to 12 months to help you continue to lose weight and make lifestyle modifications. In addition, more than 12 months of coaching will give you the training and support so that you learn how to keep the weight off for the long term.

What do I do if I gain weight?

Many people are able to use the time with their ORBERA365™ Balloon and coaching to develop a new lifestyle so that they are able to maintain or possibly continue to lose weight. This is why the coaching and follow up after the balloon removal is so important. However, some patients may regain weight after the balloon is removed.

“I lost **14 kg** with ORBERA® and kept the weight off for **5 years.**”

Thanks to ORBERA®, Andrew lost 14 kg



NOTES

[illegible]

[illegible]

Important ORBERA365™ IntraGastric Balloon System Safety Information:

The ORBERA365™ IntraGastric Balloon System is a weight loss aid for 1) adults with a body mass index (BMI) of 27 to 50kg/m² who failed to achieve and maintain weight loss with a supervised weight-control program or 2) adults with a BMI greater than 40 kg/m² or BMI of 35 kg/m² with comorbidities, prior to obesity or other surgery, in order to reduce surgical risk.

To receive ORBERA365™ you must be willing to also follow a 12-month program that includes a healthy diet and exercise plan. If the diet and exercise program is not followed, it is very likely that you will not experience significant weight loss results; in fact, you may not experience any weight loss.

Losing weight and keeping it off is not easy, so you will be supervised throughout this program by a team of physicians, physiologists, and nutritionists. This team will help you make and maintain major changes in your eating and exercise habits.

ORBERA365™ is placed for no more than twelve months. Balloon placement in the stomach for longer than 12 months may put you at increased risk for complications.

Some patients are ineligible to receive ORBERA365™. Your doctor will ask you about your medical history and will also perform a physical examination to determine your eligibility for the device. Additionally, at the time of placement, the doctor may identify internal factors, such as stomach irritation or ulcers, which may prevent you from receiving ORBERA365™.

You must not receive ORBERA365™ if you are pregnant, a woman planning to become pregnant within twelve months' time, or breast-feeding.

Complications that may result from the use of ORBERA365™ include the risks associated with any endoscopic procedure and those associated with the medications and methods used in this procedure, as well as your ability to tolerate a foreign object placed in your stomach. Possible complications include: partial or complete blockage of the bowel by the balloon, insufficient or no weight loss, adverse health consequences resulting from weight loss, stomach discomfort, continuing nausea and vomiting, early balloon removal, abdominal or back pain, acid reflux, influence on digestion of food, blockage of food entering the stomach, bacterial growth in the fluid filling the balloon which can lead to infection, injury to the lining of the digestive tract, stomach or esophagus, and balloon deflation.

Important: For full safety information please talk with your doctor.

*Results are from individual customer testimonials and your results may vary.

MKT-01145-01R01



**ARE YOU READY TO START
LOSING WEIGHT WITH
ORBERA365™?**

GET STARTED TODAY

www.orbera.com/UK