

THE 100% NON-SURGICAL SOLUTION

Orbera365[™]
managed weight loss system

**LOSE WEIGHT
WITHOUT SURGERY**



OVER 220,000
#1
GASTRIC
BALLOON
BALLOONS PLACED WORLDWIDE

**Lose 3x the weight of
diet & exercise alone**

Now with 12 months of support

START LOSING 3X THE

TIRED OF SPENDING MONEY ON WEIGHT LOSS PROGRAMS THAT DON'T WORK?

The ORBERA365™ weight loss system provides you with positive results in only 6 months and stays in place for up to 12 months for ongoing support.

ORBERA365™ combines the leading gastric balloon technology along with a support team of experts and a personalized diet and exercise plan all in one program.


This comprehensive program combines the portion control of the stomach balloon with the long-lasting healthy lifestyle coaching to provide effective and safe weight loss.

WEIGHT WITH ORBERA365™




HOW ORBERA365™ WORKS


THE 100% NON-SURGICAL SOLUTION

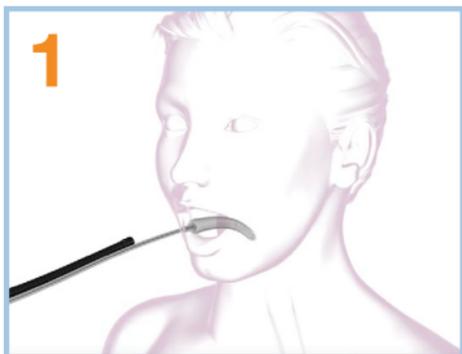

Deflated ORBERA365™ balloon is inserted through your mouth and into your stomach.

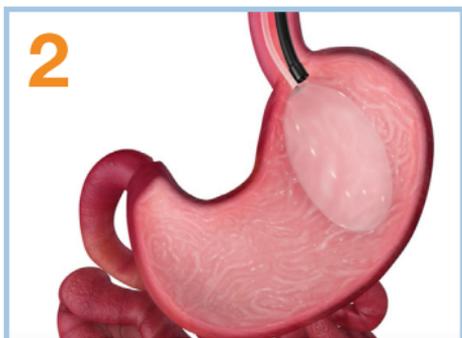


A 50cc syringe is then used to fill the balloon with a sterile saline solution to the size of a grapefruit.

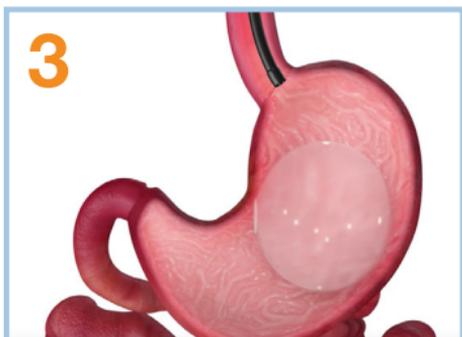

The balloon takes up enough space in your stomach to help you adapt to healthier portion sizes.

UP TO 12 MONTHS LATER

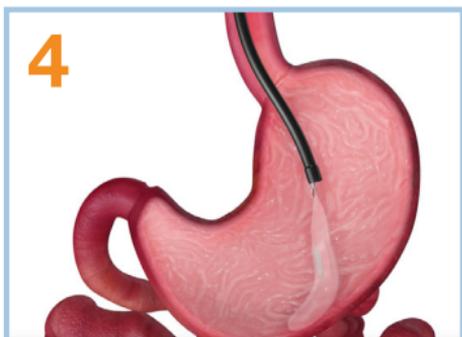

The balloon is removed 12 months later with a non-surgical procedure.


ORBERA365™ HELPS YOU FEEL FULLER AND EAT SMALLER MEALS.



**12
MONTHS**

12 MONTH PLACEMENT & ONGOING SUPPORT

Learn how to change your lifestyle over 12 months and keep the weight off with the care and support of a team of experts.

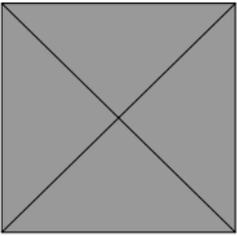

NON-SURGICAL WEIGHT LOSS

ORBERA365™ is a reversible, non-surgical weight loss procedure, which means no incisions, stitches, scars, or anatomical modifications.



220K

REAL WORLD RESULTS

For over 20 years, ORBERA365™ has helped thousands of people transform their lives and meet their weight loss goals.



#1


#1 WEIGHT LOSS GASTRIC BALLOON

Recognized for safety and effectiveness, ORBERA365™ is the global leading weight loss balloon.

BALLOON DESIGN PERFECTED OVER 20 YEARS

ORBERA365™ is the #1 gastric balloon around the world based on a reputation of quality and reliability. Key features of the ORBERA365™ gastric balloon contribute to its excellent standing and years of success. You can feel confident that your gastric balloon has a design focused on your safety and weight loss.



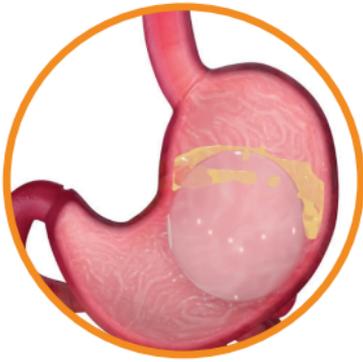

Effective Weight Loss

Designed to take up space in the stomach to help improve portion control and help you feel full. The balloon size and shape have been shown to help patients lose 3x the weight of diet and exercise alone.



Ultra Smooth and Round

There are no protrusions, no sharp edges and no hard surfaces that can irritate your stomach. ORBERA365™ offers an incredibly smooth, soft and round shape to put your safety first and limit the risk of stomach ulcerations.



7+ Layers of High Density Silicone

Each balloon is created with 7+ layers of some of the highest medical-grade silicone to ensure durability, so it can withstand digestive fluids and stomach acid with ease over the course of 12 months.



Optimal and Customizable Size

Tailored to your stomach, at about the size shape and size of a grapefruit, each balloon can be filled with as little as 400ml or as much as 700ml based on your stomach.

YOUR QUESTIONS ANSWERED

How much does it cost?

The total cost of the ORBERA365™ Program will vary but is typically an all inclusive price that includes the ORBERA® Balloon, Placement Procedure, Removal Procedure, Facility Fees and 18-24 months of physician and dietary coaching support.

What does it feel like?

ORBERA365™ takes up space in patients' stomachs and it will make you feel very full with a small meal. It helps you learn proper portion control over the 12 months with it in your stomach.

It typically takes a few days for your body to adjust to the ORBERA365™ balloon and most patients experience some nausea, vomiting and reflux, but this typically goes away and is managed by your physician.

Does it really work? How much can I expect to lose?

ORBERA® has helped thousands of patients around the world lose that stubborn fat. ORBERA® has been studied in over 200 clinical studies around the world. In the FDA approval study, patients on average lost 3x the weight of diet or exercise, or about 10% of their total body weight.

When will I see the results?

The weight loss is rapid with most of the weight loss occurring in the first three months after balloon placement. ORBERA365™ can be kept in place for up to 12 months to help you continue to lose weight and make lifestyle modifications. In addition, more than 12 months of coaching will give you the tools and knowledge to learn how to keep the weight off for the long term.

What do I do if I gain weight?

Many people are able to use the time with their ORBERA365™ Balloon and coaching to develop a new lifestyle so that they are able to maintain or possibly continue to lose weight. This is why the coaching and follow up after the balloon removal is so important. However, some patients may regain weight after the balloon is removed. In the US clinical study, patients on average maintained ~75% of their weight loss 6 months after the balloon was removed.

“I lost **14 kg** with ORBERA® and kept the weight off for **5 years.**”

Thanks to ORBERA®, Andrew lost 14 kg





**ARE YOU READY TO START
LOSING WEIGHT WITH
ORBERA365™?**

GET STARTED TODAY

www.orbera.com/UK

